

ABDUCTOR/CONTRACTION CUSHION



Reorder# 555060

PURPOSE

A new design for leg abduction and contraction control. Allows patient to sleep comfortably on his side or back. Easy to apply. Velcro® straps keep this cushion properly positioned for maximum effectiveness.

Application

1. Place leg separator between patient's legs. Extend Velcro® straps to left and right sides. (See illustration A)
2. Lie patient on side and place strap through D-ring. (See illustration B) Secure Velcro to strap surface. (See illustration C) Repeat on opposite side. Note: Straps should be snug enough to keep cushion securely in place.
3. Check behind patient's knees to make certain that cushion is properly positioned. (See illustration D)

Easy Care

The removable cloth cover may be machine laundered and dried. The vinyl cushion cover cleans with soap and water.

