


# CATHETER TUBE HOLDER

Re-Order #	Previous #	Length	Unit
<input type="checkbox"/> 101080	0105	30 Inches	Dozen
<input type="checkbox"/> 101090	0105BP	30 Inches	36/Cs.

## PURPOSE

Safety secures catheter tube to patient's leg. May also be used to secure and position nasogastric tubes. A comfortable alternative to skin-irritating tape.

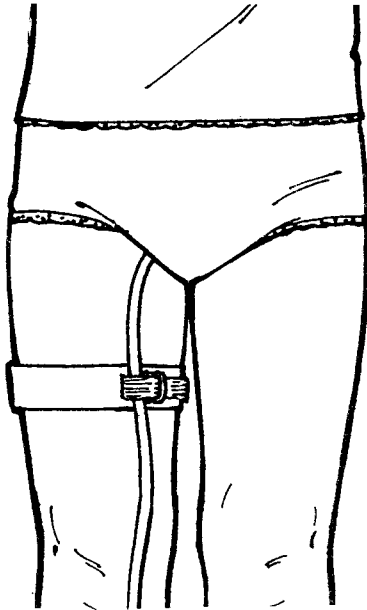

## APPLICATION

1. Wrap tube holder band around patient's leg or head as shown in illustrations. Cloth-backed side faces out. Band should be trimmed if there is more than a five-inch overlap.
2. Secure band by pressing "hook" tab against "loop" cloth backing.
3. Position tube at next to double ring. Bring "hook" material over tube and through open space in ring. Making certain that tube is snugly secured, press "hook" material against "loop" cloth backing.



## CAUTION

Do not wrap band so tightly as to impair circulation or cause patient discomfort.