



# ChairPro<sup>™</sup> / MultiPro<sup>™</sup> Gel-Foam Pad Alarm

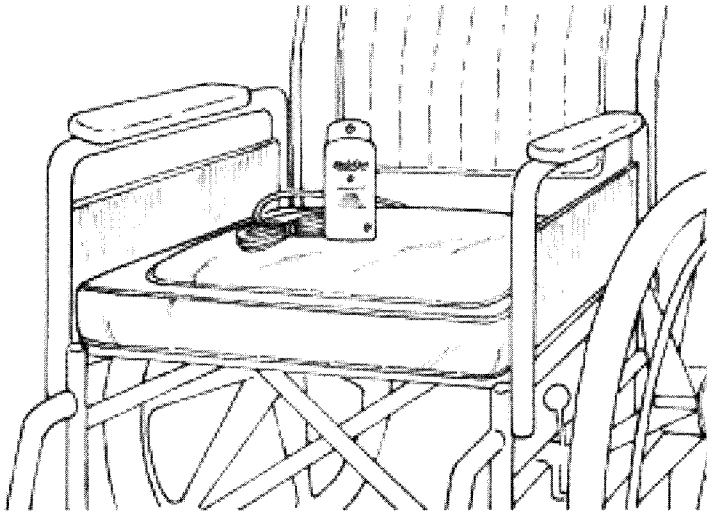


Model #	Description	Cushion Size	Guarantee
909530	Gel-Foam Pad Alarm	16" x 16" x 2"	One Year
909531	Gel-Foam Pad Alarm	18" x 16" x 2"	One Year
909532	Gel-Foam Pad Alarm	20" x 16" x 2"	One Year

## INSTALLATION

1. Place the cushion on the wheelchair with safety straps and alarm connector cord toward back.
2. Secure safety straps and place connector cord through space between wheelchair seat and backrest
3. Make certain that connector cord is away from the wheels and other moving parts
4. Attach the two "spade" connectors on the seat sensor cord to those on the enclosed with mini-phone plug. (Illustration 1)

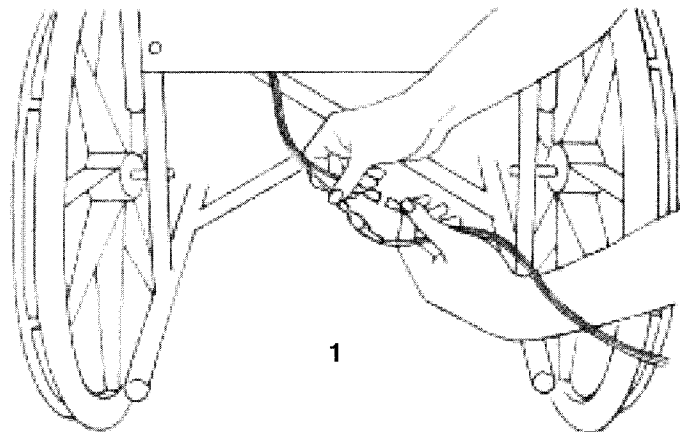

5. Insert mini-phone plug into Mini-Phone plug receptacle on right side of alarm. (Illustration 2) Alarm will sound. Press Auto-Reset to silence alarm.

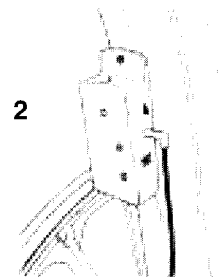

## OPERATION

1. Use the following procedure to test the system before each use.
  - a. Press Battery-Test button. If alarm is weak or fails to sound, replace 9-volt battery and press button again. If alarm is still weak or fails to sound, discontinue use of alarm.
  - b. Seat patient on cushion. Battery indicator should flash. If light fails to flash, replace battery. If battery indicator light does not flash after battery is replaced, discontinue use of alarm. (Note: The body weight of very small or frail patients may not be sufficient to activate the system. In that case, the indicator light will not flash. We recommend that you consider other MultiPro chair sensor systems for these patients.)
  - c. Assist patient to rise to the point where his/her weight is no longer on cushion. Alarm should sound. Reseat patient. The alarm will silence itself. The system is now monitoring the patient. If the alarm fails to sound, make certain that the spade connectors and mini-phone plug are secure. Retest. If alarm still fails to sound when patient's weight is removed from the cushion, discontinue use of system.
2. Select high or low volume.
3. Alarm is activated when the patient leaves or falls from the chair. Press the Auto-Reset button to silence the alarm. It is now in stand-by mode. Alarm will automatically reset itself and go into monitoring mode when patient is resealed on the cushion. **There is no on/off switch.**