
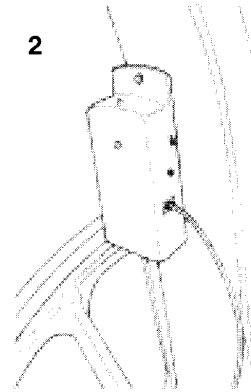


ChairPro™ / MultiPro™ with Wheelchair Sensor Pad

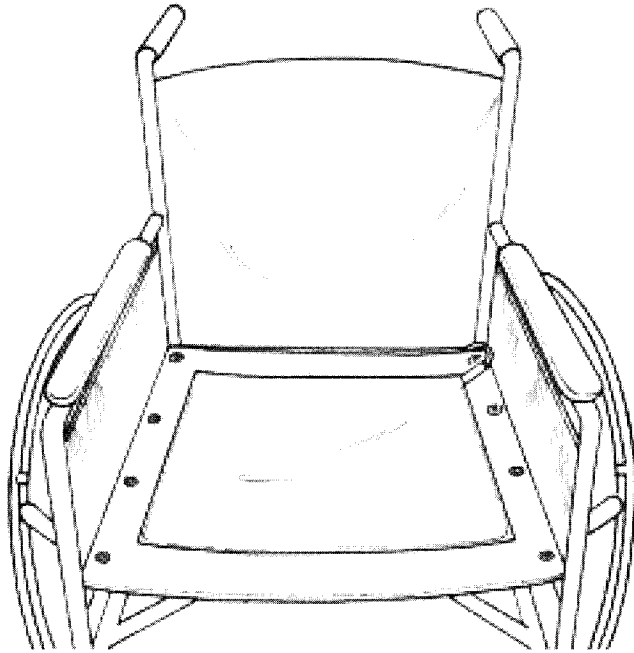


3. Insert telephone-style plug into Modular Receptacle on right side of alarm. (Illustration 2) Alarm will sound. Press Auto-Reset to silence alarm.

Model #	Guarantee	Unit
909360	45 Days	Each
909361	45 Days	5/Pk
909362	45 Days	10/Pk
909363	180 Days	Each
909364	180 Days	5/Pk
909365	180 Days	10/Pk

INSTALLATION

1. Place sensor pad on wheelchair seat. Connector cord to the rear of the seat. (Illustration 1).

2. Run connector cord through the space between the wheelchair seat and backrest. Make certain that the cord is away from wheels and other moving parts.

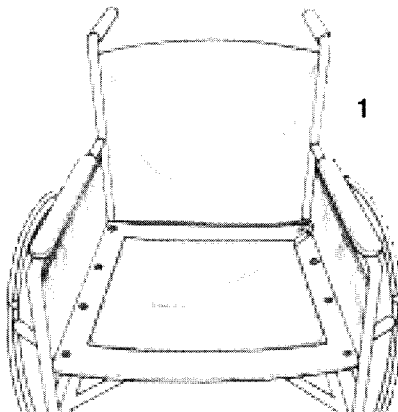

OPERATION

- 1.** Use the following procedure to test the system before each use.
 - a.** Press Battery-Test button. If alarm is weak or fails to sound, replace 9-volt battery and press button again. If alarm is still weak or fails to sound, discontinue use of alarm.
 - b.** Press your palm firmly against sensor pad. Battery indicator light should flash. If light fails to flash, replace battery and/or sensor pad. If battery indicator light does not flash after battery and/or sensor pad are replaced, discontinue use of alarm.
 - c.** Remove your hand from sensor pad. Alarm should sound. Press Auto-Reset button to silence alarm. If alarm fails to sound, check plug connection. If plug is properly connected, replace battery and/or sensor pad. Retest. If alarm again fails to sound, discontinue use of system.
- 2.** Select high or low volume.
- 3.** Seat patient on pad. The system will automatically go into monitoring mode. The alarm will sound if patient leaves or falls from chair. Press Auto-Reset button to silence alarm. The alarm is now in stand-by mode. **There is no on/off switch.**

Important Notices

- 1.** If patient/resident is using a wheelchair cushion, the sensor pad must be placed on the cushion, not under it. This system will not work if the sensor pad is placed beneath a cushion.
- 2.** Sensor pad may be cleaned with a damp cloth and mild cleanser. Do not immerse pad in water or cleaning solution. Do not sterilize.
- 3.** Inspect pad before each use. Replace if damaged.
- 4.** Read "Warnings" on sensor pad.