




# ChairPro™ SENSOR PAD ALARM

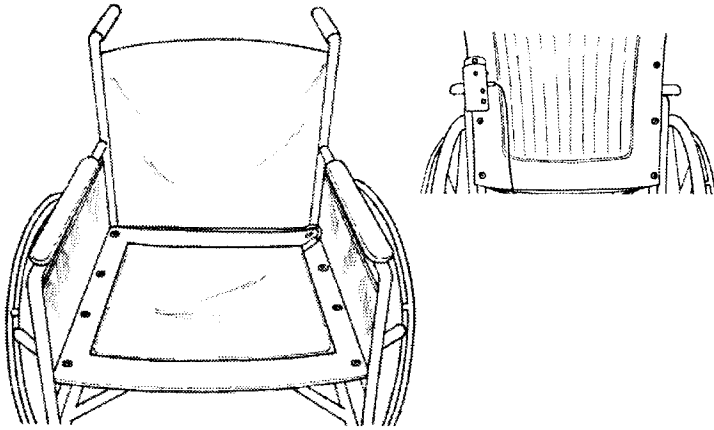

Model #	Description	Unit
909350	ChairPro Sensor Pad Alarm w/45-day sensor pad	Each
909355	ChairPro Sensor Pad Alarm w/180-day sensor pad	Each

See reverse side for replacement sensor pad ordering information.

## GUARANTEES

1. The ChairPro Safety Alarm Unit is replacement guaranteed for one year following the date of purchase.
2. The ChairPro sensor pads are replacement guaranteed for either 45 days or 180 days (see guarantee imprint on sensor pad) following the date they are put into service. That date must be noted on the pad for this guarantee to be effective.

These guarantees cover defects in materials and workmanship under conditions of normal use. Please return defective part to dealer from whom it was purchased. The dealer will send the defective part to Skil-Care and arrange for shipment of a replacement.

## PURPOSE


The ChairPro Sensor Pad Alarm is designed to alert caregivers when a patient/resident has left or fallen from the wheelchair.

## CAUTIONS

1. The ChairPro Sensor Pad Alarm is not a substitute for proper supervision. Fall-risk residents/patients should be monitored regularly.
2. Check battery each time alarm is used. If weak, replace the 9-volt battery.
3. LED should glow when patient/resident is seated on the sensor pad. If LED fails to glow, check battery. Replace if necessary. If LED does not glow after battery is replaced, check sensor pad. Replace if necessary.
4. If patient/resident is seated on a wheelchair cushion, place sensor on top of cushion. This system will not work if sensor is placed beneath cushion.

Alarm unit made in China. Instructions printed in USA.



## INSTALLATION Alarm Unit

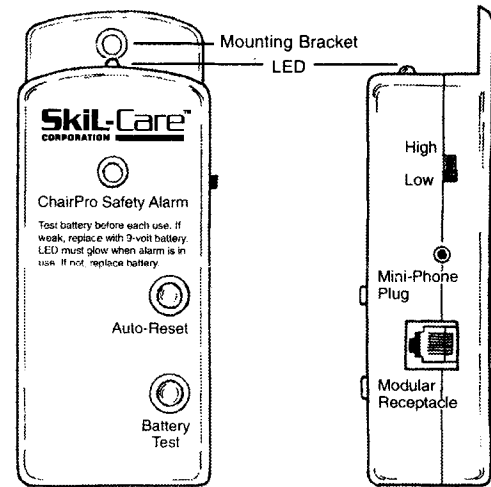

You may choose one of the following two ways to attach the alarm unit to the wheelchair.


### Mounting Bracket

1. Remove the second-from-top screw from either the left or right side of the wheelchair backrest.
2. Remove grommet from screw, place the screw through hole on alarm unit mounting bracket, and fasten screw to wheelchair backrest. (Illustration A)

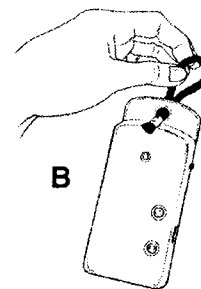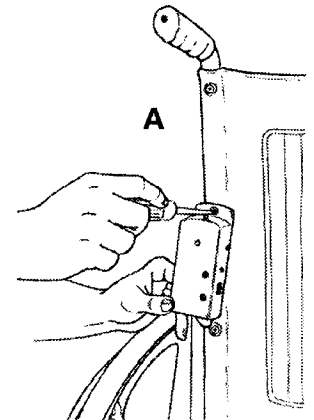

### Cord Loop

1. Insert enclosed cord loop through bracket hole. (Illustration B)

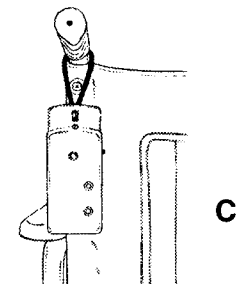


2. Hang from wheelchair push-handle. (Illustration C)

Continued on reverse side.

## Sensor Pad

1. Place sensor pad on wheelchair seat as shown (Illustration D)

2. Sensor pad connector cord is then placed through the space between the wheelchair seat and backrest. Make certain that it is away from wheels and other moving parts.

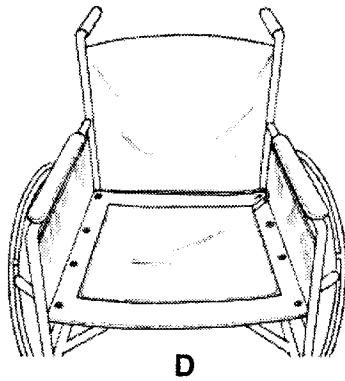

3. **Important:** If patient/resident is using a wheelchair cushion, the sensor pad must be placed on the cushion, not under it. This system will not work if sensor pad is placed beneath a cushion.

4. Sensor pad may be cleaned with a damp cloth and mild cleanser. Do not immerse pad in water or cleaning solution. Do not sterilize.

5. Inspect pad before each use. Replace if damaged.

6. Read "Warnings" on sensor pad


## OPERATION

1. Remove battery compartment cover on left side of alarm unit and install 9-volt battery.

2. Press Battery Test button. If alarm doesn't sound, check battery connection and retest. If alarm fails to sound, replace battery.

3. Insert telephone-style plug into Modular Receptacle on right side of alarm. (Illustration E) Alarm will sound. Press Auto-Reset to silence alarm.

4. Before using, test unit by pressing the palm of your hand firmly against the sensor pad. While doing this, check alarm to make certain that LED is glowing. Remove your hand. Alarm should sound. Press Auto-Reset button to silence alarm. If alarm fails to sound, make certain that telephone-style plug is secure.

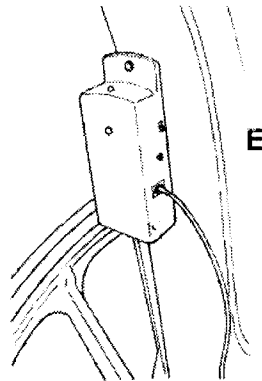

5. Select High or Low alarm volume using switch on right side of alarm.

6. When alarm is activated by patient/resident leaving the wheelchair, it may be silenced by pressing the Auto-Reset button. The alarm is now in stand-by mode. The system will automatically go into the monitoring mode when the patient/resident is resealed. **There is no on/off switch.**

## SAFETY REMINDERS

1. Check battery before each use. If alarm is weak or fails to sound, replace 9-volt battery.

2. When unit is in use, check LED regularly. If it is not glowing, replace 9-volt battery.

**Replacement sensor pads are available from your medical supply dealer.**

Model #	Guarantee	Model #	Guarantee	Unit
909360	45 Days	909363	180 Days	Each
909361	45 Days	909364	180 Days	5/Pk
909362	45 Days	909365	180 Days	10/pk

All ChairPro sensor pads are 7"x 15".

**Skil-Care**<sup>™</sup>  
CORPORATION

Advanced design for nursing and therapy.<sup>™</sup>

Alarm unit made in China. Instructions printed in USA.

Visit us online  
at [Skil-Care.com](http://Skil-Care.com)