


# DOOR STRIP ALARM

## Reorder #

909217

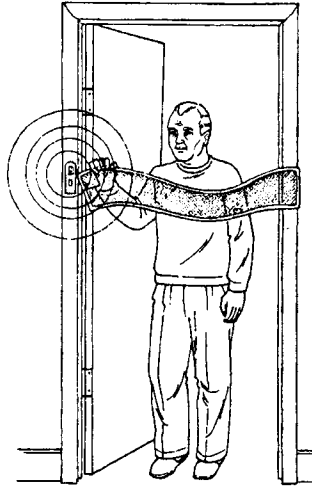

### Purpose

The brightly colored strip directs the wanderer away from hazardous areas and other residents' rooms. It also acts as a reminder for the fall-risk resident to request assistance before leaving his room. If a wanderer proceeds through a doorway, the alarm is activated. The Door Strip Alarm operates on a standard 9-volt battery (included) and requires no electrical installation. Easy-release Velcro® attaches the Door Strip Alarm to the doorframe to create a visual, not a physical barrier.

### Warning

**Do not place the Door Strip Alarm on fire or emergency exit doors.**

### Instructions

1. Install 9-volt battery. Battery compartment is on side of alarm unit.
2. The Door Strip Alarm package includes a **long** and a **short** Velcro® strip. Peel the backing off the **short** strip and attach it to the **left** side of the doorframe. Attach the **long** strip to the **right** side of the doorframe. Strips should be attached at the resident's shoulder height. See illustration A.
3. Attach alarm unit to the **short** strip on the **left** side of the door. See illustration B.
4. Attach the Velcro® material on the door strip to the **long** strip on the **right** side of the door. Bring door strip across the door and attach the magnet sewn into the door strip to the strip on the alarm unit. See illustration C.
5. Set alarm volume by moving the switch to the HI or LOW position.

### Important

Test the Door Strip Alarm before each use. Move the switch to the HI position and remove the magnet. If alarm is weak or does not sound, replace the 9-volt battery.

