


## Extended Armrest Pads

Reorder #	Description	Unit
<input type="checkbox"/> 703130	Full-Arm Wheelchair (16")	Pair
<input type="checkbox"/> 703131	Half-Arm Wheelchair (11")	Pair

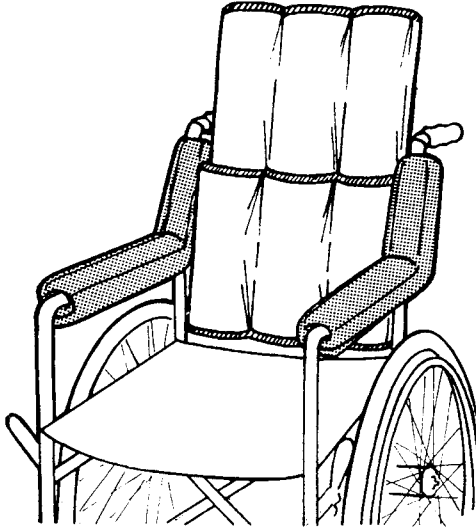

### PURPOSE

Provides foam-padded cushioning for full length of wheelchair arms. Recommended for use with wheelchairs fitted with Skil-Care's Reclining Wheelchair Backrest or with any wheelchair that requires additional padding on the armrests.

### APPLICATION

Place long section of pad over armrest and short section over section that connects armrest to back frame. Secure Velcro® beneath armrest. There is no need to remove existing armrest pads.

### LAUNDERING

The armrest pads may be washed and dried at temperatures up to 180°F (82°C). Do not use bleach. To extend the useful life of the Velcro® closures, secure the "hook" material to the "loop" material when laundering.