

FIREGARD™ TRAY COVER




Reorder #
□705060

Purpose

The Firegard Tray Cover is designed to protect Skil-Care's SofTop Wheelchair Tray from burns caused by cigarettes and matches. It fits both the nylon covered (#705019) and vinyl covered (#705020) models, which are used on 16" - 18" wheelchairs. The Firegard Tray Cover meets NFPA 701.

Application

1. Insert back of tray (padded side up) into cover and bring Velcro® bands and around-the-back straps through the cutouts. (Illustration A)
2. Turn tray over and bring cover over front of tray. (Illustration B)

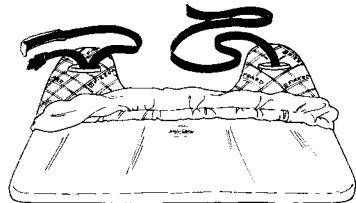


A

Easy to Care

The Firegard Tray Cover is liquid-proof and may be cleaned with a spray-type cleanser and a damp cloth.

Important Note

Lit cigarettes and burning matches that come in contact with the tray cover may cause it to discolor. This in no way reduces the fire-retardant qualities of the fabric.

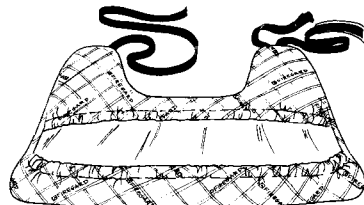

B