


FOOT STABILIZER

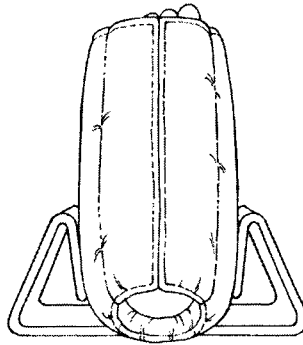

Reorder # Size

- 503038 Large/Bariatric
- 503039 Small/Medium

Purpose

Helps prevent foot rotation. Recommended for hip-fracture patients. May be used with any of Skil-Care's Heel-Float heel suspension boots, the Ultra-Soft Heel Cushion, and the Gel-Foam Heel Cushion. Skil-Care's Foot Stabilizer may also be used with heel suspension boots and fiber-filled heel cushions made by other manufacturers.

Application

1. Place heel pressure relief device (foot suspension boot or fiber-filled heel cushion) on patient.
2. Insert device into Foot Stabilizer as shown in illustration above. Fits right or left foot.
3. The Foot Stabilizer is most effective when positioned at the ankle.

Easy Care

The durable, reusable Foot Positioner may be cleaned with spray-type cleanser.