


FOOT STABILIZER

Reorder # 503039

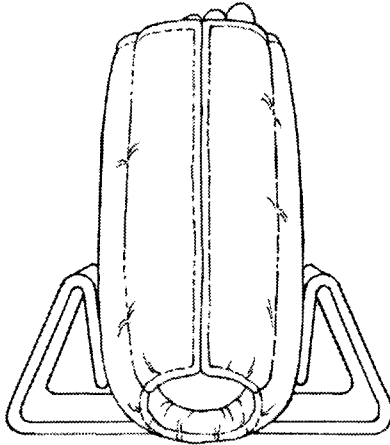

Purpose

Use with Skil-Care's Heel Float, Ultra-Soft Heel Cushion, Gel-Foam Heel Cushion, or heel suspension boots and fiber-filled heel cushions from other manufacturers to prevent foot rotation. Recommended for hip fracture patients.

Application

1. Place heel pressure relief device (foot suspension boot or fiber-filled heel cushion) on patient.
2. Insert device into Foot Stabilizer as shown in illustration above. Fits right or left foot.
3. The Foot Stabilizer is most effective when positioned at the ankle.

Easy Care

The durable, reusable Foot Positioner may be cleaned with spray-type cleanser.