


# Gel-Foam Heel Cushion

Reorder #	Description	Unit
<input type="checkbox"/> 503025	Gel-Foam Heel Cushion	Pair

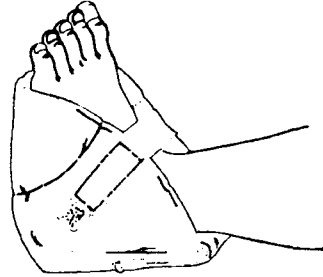

## An improved design for pressure sore prevention.

The Gel-Foam Heel Cushion combines comfortable, resilient foam with gel to relieve heel pressure. The water-based gel insert cools to reduce skin damaging perspiration. Covered with a brushed polyester fabric that prevents skin shearing and friction, the Gel-Foam Heel Cushion is durable and completely launderable.

## Easy to Use

Place on patient's foot making certain that gel pad is positioned beneath heel. Secure Velcro® closure.

## Easy Care

Remove gel insert before laundering. (See illustration.) Wash and dry at temperatures that do not exceed 180°F (82°C). Do not use bleach.

