

## Gerisleeves

Reorder	Description	Size	Unit
<input type="checkbox"/> 503355	Gerisleeve/Arm	Small	Pair
<input type="checkbox"/> 503350	Gerisleeve/Arm	Medium - Large	Pair
<input checked="" type="checkbox"/> 503360	Gerisleeve/Leg	Universal	Pair




### Purpose


Gerisleeves provide comfortable protection for sensitive skin. They shield arms and legs against skin damage in the bed or wheelchair, and protect I.V. and wound sites.

### Application

**Arms** – Select correct size for patient. Place sleeve over arm. Thumb-hole cutouts keep sleeves in place while permitting full use of all fingers. Be certain that sleeves extend above the elbows for maximum skin protection. (Illustration A.)

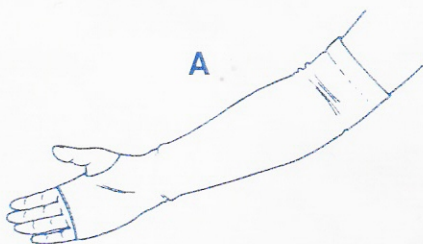

**Legs** – Leg sleeves are universally sized. For wheelchair use, Gerisleeves should extend from ankles to above calves. Fold excess material to form cuffs at ankles. (See illustration at top of sheet.) For bed use, extend sleeves beyond heel to provide protection against skin shearing. (Illustration B.)

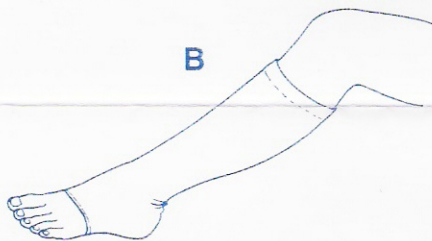

### Laundering

Gerisleeves may be machine washed and dried at temperatures that do not exceed 180° F (82° C). Do not use bleach.