




LOCATOR ALARM

Reorder #909210

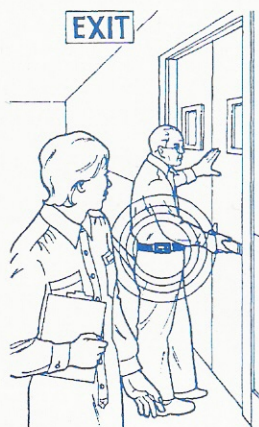

Purpose

The Locator Alarm helps you locate and identify a wandering resident. Standard personal alarms stay attached to the wheelchair when the resident exits it. The Locator Alarm stays with the wandering resident to help you find him quickly. The resident wears the Locator Alarm which alerts any staff member he passes that the resident is a wanderer and requires attention. Skil-Care's Locator Alarm is secured inside a pouch that makes it inaccessible to most residents. The alarm and the pouch are carried on a belt designed to conceal the buckle and reduce the possibility that the resident will open it. Use Skil-Care's Locator Alarm as part of your fall and elopement prevention programs.

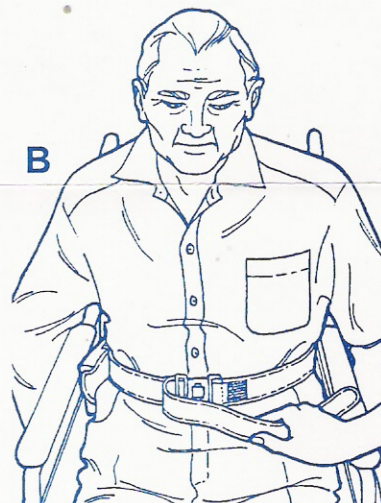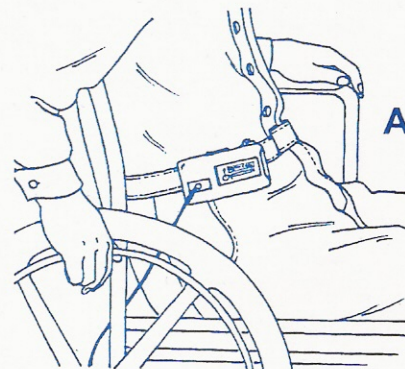

Important

Test the Locator Alarm before each use. Move the switch to the HI position and remove magnet. If alarm signal is weak or does not sound, replace the 9-volt battery. To replace battery, remove alarm from pouch and slide off battery compartment cover.

Instructions

1. Set alarm volume by moving the switch to the HI or LOW position.
2. Place strap around resident's waist. Secure the buckle and adjust the strap so that it fits snugly. Position alarm at resident's side. **IMPORTANT: The magnetic pull-switch must face the back of the wheelchair.** (Illustration A)
3. To conceal the buckle, bring loose end of strap OVER the buckle (Illustration B) and secure Velcro[®] closure. (Illustration C) Suggestion: You may position the buckle behind the resident for additional concealment.
4. Place the cord behind the resident and through the space between the seat and backrest. (Illustration D)
5. Place the loop over the kickspur. (Illustration E) Secure the cord by pressing button on cord lock and pulling cord through. (Illustration F)
6. To silence alarm once it has been activated, move switch to OFF position.

