


OVERLAP WHEELCHAIR BELT

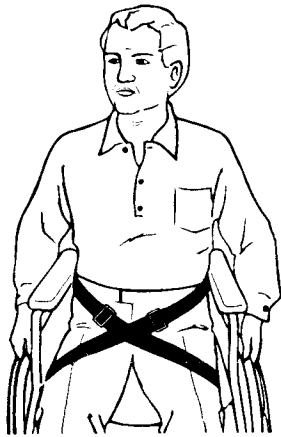

| Reorder # | Size | Unit |
|-----------|-----------|------|
| 701040 | Universal | 1 |

Caution: Federal law restricts this device to sale by or on the order of a physician or licensed health care professional.

Indications - The Skil-Care Overlap Wheelchair Belt is recommended for use by a patient who:

- Requires a safety belt while seated in the wheelchair
- Requires a belt that controls sliding down on the wheelchair seat
- Requires a reminder not to leave the wheelchair without assistance

Contraindications for Use - Do not use this device for a patient who:

- Has a history of removing belt-style, vest-style, and poncho-style restraints
- Requires upper body posture support
- Is in a geri-chair or lounge chair. This device is intended for wheelchair use only.

Installation

1. Remove the front screws from the left and right sides of the wheelchair seat.
2. Using the screws from the wheelchair seat, attach both sections of the Overlap Belt as shown in illustration A. Make certain that the screws are tight.

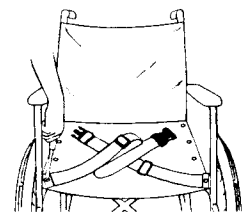


Illustration A

Application

1. Before seating patient, make certain that both brakes are locked and both footrests are positioned to the sides of the chair. Seat the patient as far back in the wheelchair as possible. Position footrests to the front of the chair and place patient's feet on them.
2. Criss-cross the belt over the patient's lap as shown in the illustration on top of this sheet.
3. Secure the buckle behind the chair as shown in illustration B. Use the sliders to adjust the belt so that it fits snugly. Note: Belt should not be so snug as to interfere with patient's breathing. You should be able to slide your open hand between the device and the patient.

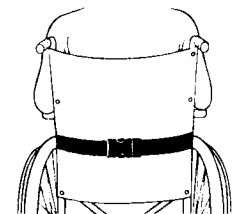


Illustration B


See **important warnings and cautions** on reverse side.

Cleaning


This device should not be laundered. It is easily cleaned with a spray-type cleanser and a damp cloth or sponge. See **important warnings and cautions** below.

Always...


Always inspect condition of device.

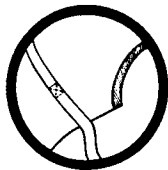

Always read package insert to determine if device is intended for wheelchair and/or bed use.

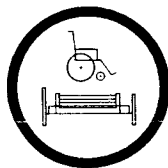

Always monitor patients in restrictive devices. Make certain to follow your facility's procedures for periodic release, toileting, and exercise.

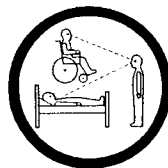

Never...

Never use a device that has worn or frayed fabric or webbing. Discard damaged units immediately. Do not alter products.



Never use a Skil-Care device as a restraint in a motor vehicle.



Never place a patient wearing a restrictive device in a place where he/she is not easily monitored.



(Date of issue: 9/3/96)