



SHOWER/TOILET SAFETY BELT


Re-Order #	Previous#	Unit Quantity
□909110	5200	Each

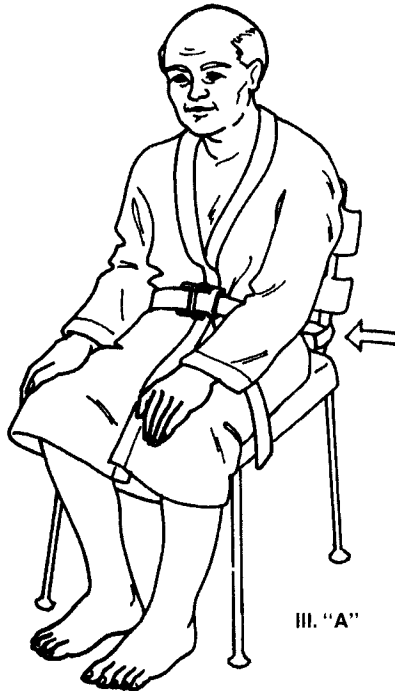

III. "A"

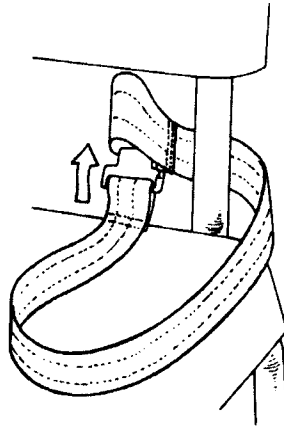

Detail

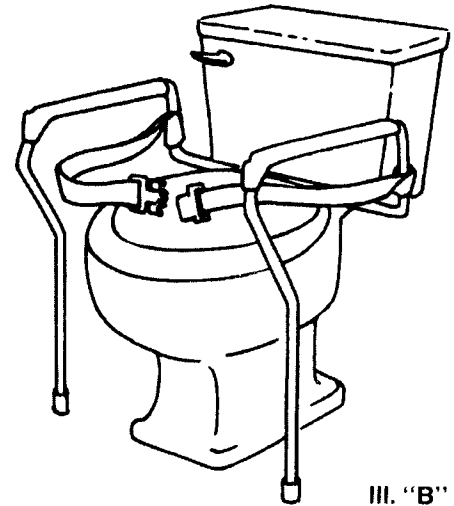

III. "B"

Makes showering or bathing your patients safer and easier. Adds security to toilet safety frames. Designed to be used with all shower/bath seats and toilet safety frames, this two-inch wide nylon belt features a durable Delrin buckle that's quality engineered for thousands of problem-free openings. Manufactured without metal or natural fiber components, the Shower/Toilet Safety Belt is rust-proof and mildew-resistant.

APPLICATION

1. The Shower/ Toilet Safety Belt secures to the vertical metal backrest supports of the shower/bath seat (See illustration "A") or the metal frame of the toilet safety device (See illustration "B").
2. Place the back (looped end) of one belt section around the metal bar on the left side of shower seat or safely frame and thread the buckle component through the loop (See illustration "C"). Pull the webbing through the loop taking up all slack. Repeat this on the right side using the other belt section.

3. Assist patient onto the shower or toilet seat. Bring belt sections in front of patient and insert male end into female end. The buckle is secure only when you hear a click.

4. To adjust belt, pull loose end until belt fits snugly around patient's waist. Do not overtighten. Belt is released by simultaneously pressing the two tabs on the top and bottom of the buckle.

CAUTION!

Do not leave your patient unattended in the shower or while he/she is using the commode or toilet. This belt has a buckle that is easily opened by patients and should not be considered as a restraining device nor as a substitute for your personal supervision.