



Universal Alarm Mounting Bracket and Belt

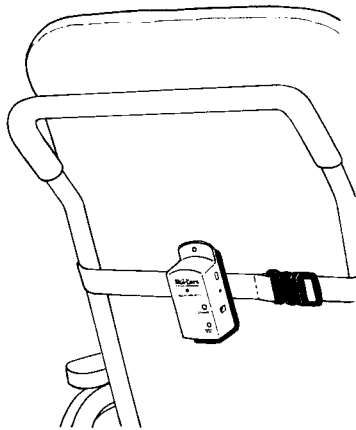

Reorder #	Size	Unit
□909201	Fits all hospital-style beds, wheelchairs, geri-chairs, lounge and other chairs.	Each

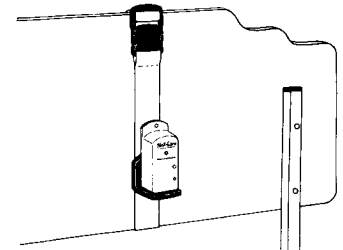

Purpose

Skil-Care's Universal Alarm Mounting Bracket Belt enables caregivers to move multipurpose alarms from the bed to the wheelchair. It also permits bed alarms to be mounted to the back of the headboard to discourage patient tampering.

Application (Chair)

1. Place belt around geri-chair, wheelchair, or other chair backrest. Secure buckle and tighten belt.
2. Remove backing from the enclosed Velcro^R strip and attach strip (hook material) to back of alarm unit.
3. Secure alarm unit to loop material on bracket. (Illustration A)

Application (Bed)

1. Place belt vertically around headboard and adjust bracket so it is aligned vertically with belt. Secure buckle and tighten belt. (Illustration B)
2. Remove back from the enclosed Velcro^R strip and attach strip (hook material) to back of alarm unit.
3. Secure alarm unit to loop material on bracket.

Application (Skil-Care BedPro or FloorPro Alarm Units)

There is no need to use the enclosed Velcro^R strip in order to secure Skil-Care's BedPro or FloorPro alarm units to the bracket. Both these units have wire brackets that permit them to be mounted to the Universal Alarm Mounting Bracket and Belt using the following procedure:

1. Place the base (wide part) of universal bracket through the alarm unit's wire bracket. (Illustration C)
2. Tilt top of universal bracket up as you continue to place it through wire bracket. (Illustration D)
3. When universal bracket is fully inserted within the wire bracket, turn universal bracket around and place alarm unit on base. (Illustration E)
4. Reverse this procedure to remove alarm unit from universal bracket.

