


VISCO-FOAM ELBOW PAD

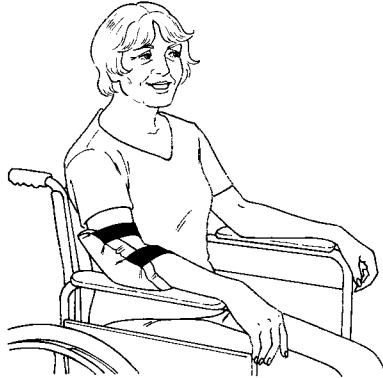

Reorder #	Unit
<input type="checkbox"/> 503230	Pair
<input type="checkbox"/> 503240	36 Pr/Cs

Purpose

Visco-Foam Elbow Pads protect sensitive skin against breakdown caused by pressure. They also guard against skin tears resulting from contact with the wheelchair. Cushioned with temperature-sensitive memory foam, these pads are as comfortable as they are protective. The Low-Shear II cover reduces friction and shear.

Easy to Use

Place the pads over the patient's elbows. The stockinet bands keep the pads in place. (See illustration above.)

Easy Care

Visco-Foam Elbow Pads are completely launderable. They may be washed and dried at temperatures that do not exceed 180°F (82°C). Do not use chlorine bleach.